- Ⓓ **Sprechende Glaswaage**
Gebrauchsanleitung
- Ⓕ **Pèse-personne vocal en verre**
Mode d'emploi
- Ⓖ **Speaking glass scale**
Instruction for Use
- Ⓔ **Báscula de vidrio con voz**
Instrucciones para el uso

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Dear Customer,

Thank you for choosing one of our products. Our name stands for high-quality, thoroughly tested products for the applications in the areas of heat, weight, blood pressure, body temperature, pulse, gentle therapy, massage and air.

Please read these instructions for use carefully and keep them for later use, be sure to make them accessible to other users and observe the information they contain.

Best regards,
Your Beurer Team

1. Interesting facts

Weighing correctly

If possible, always weigh yourself at the same time of day (ideally in the morning), after going to the toilet, on an empty stomach and without clothes to achieve comparable results.

2. Important notes – keep for later use!

■ Caution – For your safety

- Attention! Do not step onto the scale with wet feet or when the scale's surface is damp -danger of slipping!
- You should protect the scale against impacts, moisture, dust, chemicals, major temperature fluctuations and keep it away from heat sources (ovens, radiators).
- Repairs may only be performed by Customer Service or authorized dealers. However, always check the batteries and replace them if necessary prior to making a complaint.
- All scales comply with EC Directive 2004/108 + supplements. Should you still have questions regarding the use of our products, please contact your dealer or Customer Service.
- The scale should occasionally be cleaned with a damp cloth. Do not use aggressive cleaning agents and never submerge the scale.
- The device is only for private use and is not intended for medical or commercial purposes.
- Please dispose of the appliance in accordance with the Directive 2002/96/EC on Waste Electrical and Electronic Equipment (WEEE). In case of queries, please contact the municipal authorities responsible for waste disposal in your area.

3. Voice function

The device offers two languages:

- Art.-Nr.: German (DE) and French (FR)
- Art.-Nr.: English (GB) and Spanish (ES)

In order to change the language selection, set the button on the top edge of the scale to the required language.

Volume adjustment: You can adjust or switch off the volume of the language function as desired by turning the dial on the upper edge of the scale:

- OFF** = switch off
- = quieter
- + = louder

4. Use

This scale is used to determine the weight of human beings. The scale has a weighing range up to 330 lbs (150 kg).

4.1 Batteries


If present, pull the battery insulating strip off the battery compartment cover or remove the battery's protective film and insert the battery according to the polarity. If the scale fails to operate, remove the battery completely and insert it again. Your scale is equipped with a "replace battery" indicator. When the scale is operated with a battery which is too weak, "Lo" appears in the display field and the scale automatically switches off.

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The battery must be replaced in this case (3 x AAA 1,5V). The used, completely drained standard and rechargeable batteries must be disposed of in specially marked collection containers, at toxic waste collection points or electrical product retailers. You are legally obligated to dispose of the batteries.

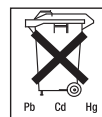Note: Batteries containing pollutants are marked with the following symbols: Pb = Battery contains lead, Cd = Battery contains cadmium, Hg = Battery contains mercury. Place the scale on a firm, flat surface; a firm floor covering is a basic requirement for correct weight measurement.

Note: On delivery the scale is set to the unit "kg". If the scale is used with the English language setting the units can be changed with the switch on the top of the scale. Select "kilograms" (kg), "pounds" (lb) or "stone" (st).

4.2 Measuring weight

Place the scale on a firm level floor (no carpet); a firm floor covering is required for correct measurement.

Audibly, briefly and firmly tap the scale with your foot to active the vibration sensor!

The entire display (Fig. 1) appears up to "0.0" (Fig. 2) as a self-test (tap-on technology).

Fig. 1

Fig. 2

Display	DE	FR	GB	ES
8888	Beep	Beep	Beep	Beep

Now the scale is ready to measure your weight.

Display	DE	FR	GB	ES
0.0	Das Gerät ist betriebsbereit	L'appareil est prêt à fonctionner	The device is ready for use	El aparato está preparado para el servicio.

Now step onto the scale. Stand still on the scale with your weight distributed evenly between both legs. The scale immediately begins to measure your weight. After flashing twice the final measurement result appears on the display. (Fig. 3)

Fig. 3

There is also an acoustic output. The scale switches off 10 seconds after you step down from it. The measuring result remains visible during this time.

If the scale detects an error during weighing, „----“ or „Err“ appears in the display (Fig. 4+5).

Display	DE	FR	GB	ES
---- / Err	Fehler	Erreur	Error	Error

Fig. 4

Fig. 5

If a weight of more than 150 kg is placed on the scale, „OvEr“ is displayed (Fig. 6).

Display	DE	FR	GB	ES
OvEr	Überlastung	Surcharge	Overload	Sobrecarga

Fig. 6

Make sure that you always switch on the scale first and wait for the display „0.0“ (Fig. 2) before stepping on.

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