beurer

BM 93 Cardio

DE Blutdruckmessgerät mit EKG-Funktion Gebrauchsanleitung	2
EN Blood pressure monitor with ECG function Instructions for use.....	32
FR Tensiomètre avec fonction ECG Mode d'emploi	59
ES Tensiómetro con función de ECG Instrucciones de uso	87
IT Misuratore di pressione con funzione ECG Istruzioni per l'uso	116
TR EKG fonksiyonlu tansiyon ölçme cihazı Kullanım kılavuzu.....	144
RU Прибор для измерения кровяного давления в плечевой артерии с функцией снятия ЭКГ Инструкция по применению	171
PL Ciśnieniomierz z funkcją EKG Instrukcja obsługi	201

CE 1639

DEUTSCH

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ENGLISH

Read these instructions for use carefully and keep them for later use, be sure to make them accessible to other users and observe the information they contain.

Dear customer,

Thank you for choosing a product from our range. Our name stands for high-quality, thoroughly tested products for applications in the areas of heat, weight, blood pressure, body temperature, pulse, gentle therapy, massage, beauty, baby and air.

With kind regards,
Your Beurer team

Table of contents

1. Included in delivery	33
2. Signs and symbols	34
3. Intended use	35
4. Warnings and safety notes	35
5. Device description	41
5.1 Main unit	41
5.2 Display description	42
6. Initial use	42
6.1 Insert the batteries	42
6.2 Settings	43
7. Usage (blood pressure)	44
7.1 Attaching the cuff	44
7.2 Adopting the correct posture	45
7.3 Select the user memory	46
7.4 Start the blood pressure measurement	46
7.5 Evaluating blood pressure measurement results	46
7.6 Saving, displaying and deleting measured values	47
8. Usage (ECG)	48
8.1 Preparing the ECG measurement	48
8.2 Measuring ECG	48
8.3 Taking the ECG measurement	49
8.4 Evaluating ECG measured values	49
8.5 Displaying and deleting ECG measured values	50
9. Transferring measurements	50
9.1 Transfer via <i>Bluetooth</i> [®] low energy technology	50
10. Cleaning and maintenance	51
11. Accessories and replacement parts	52
12. What if there are problems?	52
13. Disposal	54
14. Technical specifications	55
15. Warranty/ service	57

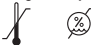
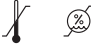
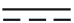
1. Included in delivery

Check that the exterior of the cardboard delivery packaging is intact and make sure that all contents are present. Before use, ensure that there is no visible damage to the device or accessories and that all packaging material has been removed. If you have any doubts, do not use the device and contact your retailer or the specified Customer Service address.

- 1x blood pressure monitor with ECG function
- 1x upper arm cuff
- 4x 1.5 V AAA batteries
- 1x storage pouch
- 1x instructions for use
- 1x quick start guide
- 1x annex for attending doctor

2. Signs and symbols

The following symbols are used on the device, in these instructions for use, on the packaging and on the type plate for the device:

	WARNING Warning notice indicating a risk of injury or damage to health		Dispose of packaging in an environmentally friendly manner
	IMPORTANT Safety note indicating possible damage to the device/accessory		Manufacturer
	Note Note on important information		European Authorised Representative for manufacturers of medical devices
	Observe the instructions for use	Storage/Transport 	Permissible storage and transport temperature and humidity
	Application part, type BF	Operating 	Permissible operating temperature and humidity
	Application part type CF	IP21	Protected against solid foreign objects 12.5 mm in diameter and larger, and against vertically falling drops of water
	Direct current		Serial number

	<p>Disposal in accordance with the Waste Electrical and Electronic Equipment EC Directive – WEEE</p>		<p>CE labelling This product satisfies the requirements of the applicable European and national directives.</p>
	<p>Do not dispose of batteries containing hazardous substances with household waste</p>		

3. Intended use

The blood pressure monitor with ECG function is used to carry out non-invasive measurement and monitoring of the arterial blood pressure values in adults and to calculate the heart rhythm. This allows you to quickly and easily measure your blood pressure, save the measured values and display the development and average values of the measured values taken. The recorded values are classified and evaluated graphically.

The ECG is used to calculate the heart rhythm. The device provides information about your average pulse value as well as any divergence from a normal ECG.

The BM 93 blood pressure monitor with ECG function provides the following functions:

- Integration of blood pressure measurement and ECG measurement into a single device.
- Measurement of systolic and diastolic pressure.
- Practical ECG measurement with integrated ECG-electrodes.
- ECG recording of 30 seconds.
- Transfer of the data via *Bluetooth*[®].

4. Warnings and safety notes

Notes on using the blood pressure monitor

- In order to ensure comparable values, always measure your blood pressure at the same time of day.
- Do not take a measurement within 30 minutes of eating, drinking, smoking or exercising.
- Before the initial blood pressure measurement, make sure always to rest for about 5 minutes.

- Furthermore, if you want to take several measurements in succession, make sure always to wait for at least 1 minute between the individual measurements.
- Repeat the measurement if you are unsure of the measured value.
- The measurements taken by you are for your information only – they are no substitute for a medical examination! Discuss the measured values with your doctor and never base any medical decisions on them (e.g. medicines and their administration).
- Do not use the blood pressure monitor on newborns or patients with preeclampsia. We recommend consulting a doctor before using the blood pressure monitor during pregnancy.
- Cardiovascular diseases may lead to incorrect measurements or have a detrimental effect on measurement accuracy. The same also applies to very low blood pressure, diabetes, circulatory disorders and arrhythmias as well as chills or shaking.
- This device is not intended for use by people (including children) with restricted physical, sensory or mental skills or a lack of experience and/or a lack of knowledge, unless they are supervised by a person who is responsible for their safety or are instructed by such a person in how to use the device. Supervise children around the device to ensure they do not play with it.
- The blood pressure monitor must not be used in connection with a high-frequency surgical unit.
- Only use the unit on people that have the specified upper arm measurement for the unit.
- Please note that when inflating, the functions of the limb in question may be impaired.
- During the blood pressure measurement, the blood circulation must not be stopped for an unnecessarily long time. If the device malfunctions remove the cuff from the arm.
- Avoid any mechanical restriction, compression or bending of the cuff line.
- Do not allow sustained pressure in the cuff or frequent measurements. The resulting restriction of the blood flow may cause injury.
- Make sure that the cuff is not placed on an arm in which the arteries or veins are undergoing medical treatment, e.g. intravascular access or intravascular therapy, or an arteriovenous (AV) shunt.
- Do not use the cuff on people who have undergone a mastectomy.
- Do not place the cuff over wounds as this may cause further injury.
- Place the cuff on your upper arm only. Do not place the cuff on other parts of the body.
- The blood pressure monitor can only be operated with batteries. Please note that data transfer and data storage is only possible when your blood pressure monitor is supplied with power. As soon as the batteries are empty, the blood pressure monitor loses the date and time.
- To conserve the batteries, the blood pressure monitor switches off automatically if you do not press any buttons for two minutes.

- The device is only intended for the purpose described in these instructions for use. The manufacturer is not liable for damage resulting from improper or careless use.

General notes on using the ECG

- The ECG measuring device is a single-channel electrocardiogram (ECG) device that you can use to rapidly record an electrocardiogram (ECG). In addition, the device provides a clear evaluation of the recording, particularly as regards any disruption of the heart rhythm.
- The ECG measuring device displays changes in the heart rhythm, which may have various different causes. These may be harmless, but may also be triggered by illnesses or diseases of differing degrees of severity. Please consult a medical specialist if you believe you may have an illness or disease.
- Electrocardiograms recorded using the ECG measuring device show the heart's function at the time of the measurement. As such, any changes occurring before or after the measurement is taken may not necessarily be detected.
- ECG measurements, such as those taken with the ECG measuring device, cannot identify all heart diseases. Regardless of the measurement taken, you should consult your doctor immediately if you experience symptoms that could indicate acute heart disease. Such symptoms could include (but are not limited to):
 - pain or pressure on the left of the chest area or abdomen,
 - radiating pain in the area of the mouth/jaw/face, or in the shoulders, arm or hands,
 - back pain,
 - nausea,
 - burning sensation in the chest,
 - tendency to collapse,
 - breathlessness,
 - rapid heartbeat or irregular heart rhythm– or, in particular, a combination of these symptoms.
- ALWAYS consult a doctor IMMEDIATELY if you experience any of these symptoms. If you have any doubts, seek an emergency medical examination.
- Do not self-diagnose or self-medicate on the basis of the measurement without consulting your doctor. In particular, do not start taking any new medication or change the type and/or dosage of any existing medication without prior approval.

- The ECG measuring device is not a substitute for a medical examination of your heart function or for medical electrocardiogram recordings, which require more complex measurements.
- It is not possible to use the ECG measuring device to diagnose illnesses or diseases that may be causing an ECG change. This is exclusively the responsibility of your doctor.
- We recommend that you record the ECG measurements and provide them to your doctor if required. This applies in particular if the status messages of the ECG measuring device do not show the “OK” symbol.

Important safety notes

- We recommend against using the device if you have a pacemaker or other implanted devices. Follow the advice given by your doctor, if applicable.
- Do not use the device with a defibrillator.
- Do not use the device during an MRT examination.
- Do not expose the device to static electricity. Always ensure that you are free of static electricity before operating the device.
- Never submerge the device in water or other liquids. Do not clean the device with acetone or other volatile solutions. Clean the device with a cloth dampened with water or a mild cleaning liquid. Then dry the device using a dry cloth.
- Do not place the device in pressure vessels or gas sterilisation devices.
- Do not allow the device to fall and do not step on or shake the device.
- Do not dismantle the device, as this could cause damage or malfunctions or impede the operation of the device.
- Do not use the device with persons with sensitive skin or allergies.
- This device is not intended for use by people (including children) with restricted physical, sensory or mental skills or a lack of experience and/or a lack of knowledge, unless they are supervised by a person who is responsible for their safety or are instructed by such a person in how to use the device. Supervise children around the device to ensure they do not play with it.
- This device is not approved for use with children weighing less than 10 kg.
- Do not allow the electrodes of the device to come into contact with other conductive parts (including earth).
- Do not store the device in the following locations: locations in which the device is exposed to direct sunlight, high temperatures or levels of moisture, or heavy contamination; locations near to sources of water or fire; or locations that are subject to strong electromagnetic influences.


- Do not inflate the arm cuff when it is not wrapped around your arm.
- Do not strongly bend the arm cuff or air tube.
- Do not inflate the arm cuff over 299 mmHg.
- Do not splash water on the arm cuff or immerse it in water.
- Do not use this device on people with sensitive skin or allergies.
- Do not operate this device in a moving vehicle (car, airplane).
- Do not use mobile phones, or other devices that emit electromagnetic fields near this device which may result in incorrect operation of the device.
- Remove batteries if won't be using this device for some time, to prevent battery to expand, leak and possibly destroy the device.
- Store the device in safe place to avoid small parts swallowed by children in any circumstances.
- Under the environment temperature of 40°C, the temperature of applied part (cuff and ECG electrodes) will reach 42°C highest.
- Keep the components of the device out of the reach of infants, toddlers or children. This device contains small parts (such as batteries) that may cause a choking hazard if swallowed by infants, toddlers or children.

Notes on storage

- The blood pressure monitor with ECG function is made from precision and electronic components. The accuracy of the measured values and service life of the device depend on its careful handling:
 - Protect the device from impacts, humidity, dirt, marked temperature fluctuations and direct sunlight.
 - Do not drop the device.
 - Do not use the device in the vicinity of strong electromagnetic fields and keep it away from radio systems or mobile telephones.
 - Only use the cuff included with the delivery or original replacement parts. Otherwise incorrect measured values will be recorded.
- We recommend that the batteries be removed if the device will not be used for a prolonged period of time.

Notes on handling batteries

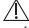
- If your skin or eyes come into contact with battery fluid, rinse the affected areas with water and seek medical assistance.
-  **Choking hazard!** Small children may swallow and choke on batteries. Store the batteries out of the reach of small children.
- Observe the plus (+) and minus (-) polarity signs.
- If a battery has leaked, put on protective gloves and clean the battery compartment with a dry cloth.
- Protect batteries from excessive heat.
-  **Risk of explosion!** Never throw batteries into a fire.
- Do not charge or short-circuit batteries.
- If the device is not to be used for a relatively long period, take the batteries out of the battery compartment.
- Use identical or equivalent battery types only.
- Always replace all batteries at the same time.
- Do not use rechargeable batteries.
- Do not disassemble, split or crush the batteries.

Notes on electromagnetic compatibility

- The device is suitable for use in all environments listed in these instructions for use, including domestic environments.
- The use of the device may be limited in the presence of electromagnetic disturbances. This could result in issues such as error messages or the failure of the display/device.
- Avoid using this device directly next to other devices or stacked on top of other devices, as this could lead to faulty operation. If, however, it is necessary to use the device in the manner stated, this device as well as the other devices must be monitored to ensure they are working properly.
- The use of accessories other than those specified or provided by the manufacturer of this device can lead to an increase in electromagnetic emissions or a decrease in the device's electromagnetic immunity; this can result in faulty operation.
- Failure to comply with the above can impair the performance of the device.


5. Device description

5.1 Main unit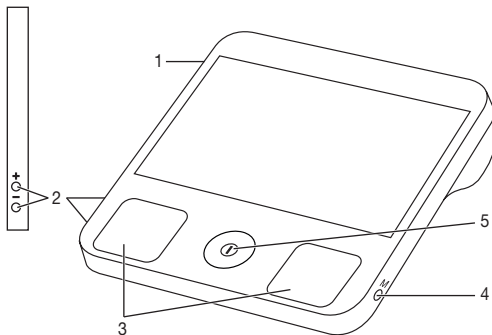

1. Connection for cuff connector
2. Setting buttons +/- (for date and time/alarm)
3. ECG Electrodes
4. Memory button **M** (to access stored measured values, change user memory)
5. **START/STOP** button  (ECG measurement / blood pressure measurement)

5.2 Display description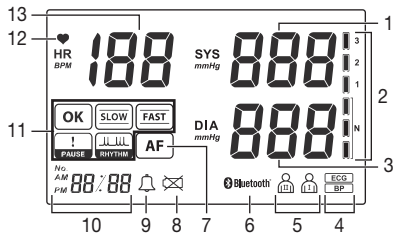


1. Systolic blood pressure display
2. Classification of measured values
3. Diastolic blood pressure display
4. Measurement mode (blood pressure = "BP", ECG = "ECG")
5. Current user memory
6. *Bluetooth*® connection symbol
7. Atrial fibrillation indicator (only for blood pressure measurement)
8. Low battery status symbol
9. Alarm symbol
10. Standby mode = time, ECG measurement = 30 sec. countdown; memory mode = shows the number of measurements/measurement time
11. ECG result display
12. Heart rate display: flashes in sync with your heartbeat during the measurement.
13. Heart rate: shows the average heart rate during the recording process.

6. Initial use

6.1 Insert the batteries

1. Open the battery compartment lid.
2. Insert four 1.5V AAA type (LR03) batteries. Make sure that the batteries are inserted the correct way round. Do not use rechargeable batteries.
3. Close the battery compartment lid again carefully.
4. The device will emit an acoustic signal and the year will flash on the display.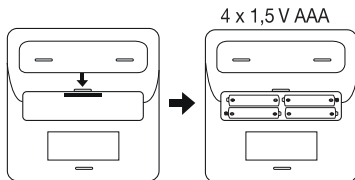

The batteries must be replaced when  appears on the display.


6.2 Settings

Set the date and time

It is essential that you set the date and time. Otherwise, you will not be able to save your measured values correctly with a date and time and access them again later.

Press and hold the function button [+] on the device when **switched off** for approx. 5 seconds. The year flashes on the display.

In this menu you can adjust the following settings in succession:

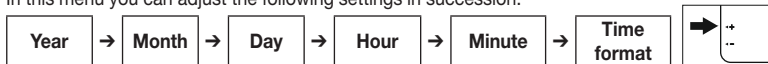
1. Set the year using the +/- buttons on the side (you can select a year between 2000 and 2099). To confirm the year, press the memory button **M**.
2. The month now flashes on the display. Set the month using the +/- buttons. To confirm the month, press the memory button **M**.
3. The day now flashes on the display. Set the day using the +/- buttons. To confirm the day, press the memory button **M**.
4. The hour now flashes on the display. Set the hour using the +/- buttons. To confirm the hour, press the memory button **M**.
5. The minute now flashes on the display. Set the minute using the +/- buttons. To confirm the minute, press the memory button **M**.
6. **24H** or **12H** now flashes on the display. Switch between the time format using +/- buttons. To confirm the time format, press the memory button **M**.
7. The set year appears on the display again. To complete the operation, press the **START/STOP** button **ⓘ**. The device then switches off.

Setting the alarm

An optional alarm can be set on the device. To set the alarm, proceed as follows:

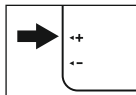1. Press and hold the - button on the side for 5 seconds. "OFF" flashes on the display if the alarm is deactivated.
2. "☞🔔" flashes on the display if the alarm is active.
3. Activate or deactivate the alarm using the +/- buttons. Press the memory button **M** to confirm the selection.


4. The alarm hour flashes on the display when the alarm has been activated. Set the alarm hour using the +/- buttons. To confirm the alarm hour, press the memory button **M**.
5. The alarm minute now flashes on the display. Set the alarm minute using the +/- buttons. To confirm the alarm minute, press the memory button **M**.
6. “” appears on the display. To complete the operation, press the **START/STOP** button . The device then switches off.

7. Usage (blood pressure)


Before commencing blood pressure measurement, observe the “Notes on using the blood pressure monitor” in chapter “2. Important notes”.

7.1 Attaching the cuff


Place the cuff onto the bare upper arm. The circulation of the arm must not be hindered by tight clothing or similar.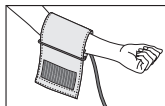


The cuff must be placed on the upper arm so that the bottom edge is positioned 2 – 3 cm above the elbow and over the artery. The line should point to the centre of the palm.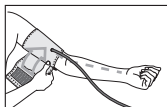


Now tighten the free end of the cuff, but make sure that it is not too tight around the arm and close the hook-and-loop fastener. The cuff should be fastened so that two fingers can fit under the cuff.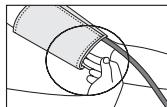

Now insert the cuff line into the connection for the cuff connector.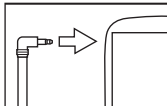

This cuff is suitable for you if the index mark (▼) is within the MIN/MAX Range after fitting the cuff on the upper arm.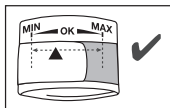


- ❶ If the measurement is performed on the right upper arm, the line should be located on the inside of your elbow. Ensure that your arm is not pressing on the line.

Blood pressure may vary between the right and left arm, which may mean that the measured blood pressure values are different. Always perform the measurement on the same arm. If the values between the two arms are significantly different, please consult your doctor to determine which arm should be used for the measurement.

Warning

The unit may only be operated with the original cuff. The cuff is suitable for an arm circumference of 23 up to 43 cm.

7.2 Adopting the correct posture

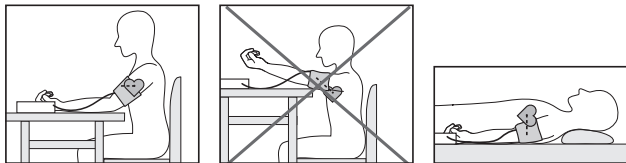- Before the initial blood pressure measurement, make sure always to rest for about 5 minutes. Otherwise deviations can occur.
- You can take the measurement while sitting or lying. Always make sure that the cuff is at heart level.
- To take your blood pressure, make sure you are sitting comfortably with your arms and back leaning on something. Do not cross your legs. Place your feet flat on the ground.

- To avoid falsifying the measurement, it is important to remain still during the measurement and not to speak.
- When taking the measurement, please keep the device lying flat (horizontally) on the table.

 Wait for at least 1 minute before taking another measurement.

7.3 Select the user memory

The device has two user memories. Each user memory can store up to 60 measured values. Once all memory spaces in a user memory have been used, the oldest measured values are replaced by the most recent.

- Briefly press the memory button **M**. The last set user is displayed. Press and hold the memory button **M** for 5 seconds to switch between the user memories.

7.4 Start the blood pressure measurement

1. As described above, attach the cuff and adopt the correct posture.
2. Press the **START/STOP** button  to turn on the device. The device is now in ECG measurement mode. You can now switch between the user memory ( / ) by press and hold the button **M** for 3 seconds. Press the **START/STOP** button  again to start the blood pressure measurement. The blood pressure symbol appears on the display and the cuff inflates. The heart symbol begins to flash as soon as the device detects a pulse.

 You can cancel the blood pressure measurement at any time by pressing the **START/STOP** button  again. Cancelled blood pressure measurements are not stored.

3. The results are displayed on the display as soon as the blood pressure measurement is complete.
4. To switch off the device, press the **START/STOP** button  or wait for 2 minutes until the device switches off automatically.

7.5 Evaluating blood pressure measurement results

Atrial fibrillation

This device can identify atrial fibrillation when measuring the blood pressure and if necessary, indicates this after the measurement with the symbol . Atrial fibrillation is defined by an irregular and often rapid heart rate increasing the risk of strokes, heart failure and other heart-related complications. Atrial fibrillation can only be determined through an examination by your doctor. If

the symbol **AF** is shown on the display after the measurement has been taken, repeat the measurement. Please ensure that you rest for 5 minutes beforehand and do not speak or move during the measurement. If the symbol **AF** appears frequently, please consult your doctor. Selfdiagnosis and treatment based on the measurements can be dangerous. Always follow your GPs instructions.

Risk indicator

The measured values can be classified and evaluated in accordance with the following table. However, these standard values serve only as a general guideline, as the individual blood pressure varies in different people and different age groups etc.

It is important to consult your doctor regularly for advice. Your doctor will tell you your individual values for normal blood pressure as well as the value above which your blood pressure is classified as dangerous.

The bar chart on the display shows which category the recorded blood pressure values fall into. If the values of systole and diastole fall into two different categories (e.g. systole in the 'High normal' category and diastole in the 'Normal' category), the graphical classification on the display always shows the higher category; for the example given this would be 'High normal'.

Blood pressure value category	Systole (in mmHg)	Diastole (in mmHg)	Action
Level 3: severe hypertension	≥ 180	≥ 110	seek medical attention
Level 2: moderate hypertension	160–179	100–109	seek medical attention
Level 1: mild hypertension	140–159	90–99	regular monitoring by doctor
High normal	130–139	85–89	regular monitoring by doctor
Normal	120–129	80–84	self-monitoring
Optimal	< 120	< 80	self-monitoring

Source: WHO, 1999 (World Health Organization)

7.6 Saving, displaying and deleting measured values

The device has two user memories. Each user memory can store up to 60 measured values. Once all memory spaces in a user memory have been used, the oldest measured values are replaced by the most recent.

You can access stored measured values on the device at any time. Proceed as follows:

1. With the device switched off, press the memory button **M**. The last stored measured value in the selected user memory is displayed. To access the measured values in the second user memory, press and hold the memory button **M** for 5 seconds.
2. To navigate between the individual measured values, press the memory button **M**. If you want to delete the entire memory of the respective user, press the memory button **M** again and press and hold it together with **START/STOP** button  for 5 seconds.

8. Usage (ECG)

8.1 Preparing the ECG measurement

Observe the following before beginning the ECG measurement:

- If the ECG-electrode surfaces are dirty, clean them with a cotton bud soaked in rubbing alcohol.
- If your skin or hands are dry, moisten them using a damp cloth before the measurement.
- Please note that there must be no skin contact between your right and left hand. Otherwise, the measurement cannot be taken correctly.
- Do not speak or move while carrying out the ECG measurement as this can cause inaccuracies during the measurement.
- When taking the measurement, please keep the device lying flat (horizontally) on the table.

8.2 Measuring ECG

The accuracy of the measurement as well as the ability to perform a measurement with this device depends on the heart configuration (shape of the heart) of each individual user. If it is not possible to take stable measurements using this device, this could have a harmless cause such as the shape of the heart. However, the cause may also be an illness or disease.

-  Please note that there must be no skin contact between your right and left hand. Otherwise, the measurement cannot be taken correctly. Stay still during the measurement, do not speak and hold the device still. Movements of any kind will falsify the measurements.

Maintain constant pressure during the measurement. Do not press the electrodes too firmly against the skin as this can cause the muscles to tense and give rise to inaccurate measured values.

8.3 Taking the ECG measurement

1. Press the **START/STOP** button  to turn on the device. The device is now in ECG measurement mode. You can now switch between the user memory ( / ) by press and hold the button **M** for 3 seconds.
2. Put your index finger and middle finger or your thumbs of both hands on both electrodes. The Measurement starts automatically if there is sufficient contact to the electrodes and the signal is stable.
 -  • The force should be kept constant during the measuring period. Do not press the electrodes too hard. Excessive muscle tension might affect the accuracy of measurement result.
 - Make sure both hand's index fingers and middle fingers or thumbs are in sufficient contact with the two electrodes.
3. A countdown on the left side of the display indicates the progress of the measurement. During the measurement, the current heart rate is displayed in real time. The symbol  will flash and the sound (if applicable) will beep in sync with your heart beat.
4. A summary of the ECG measurement and the average heart rate appear on the display after the countdown has elapsed.
5. To switch off the device, press the **START/STOP** button . Alternatively, the device will switch off automatically after 2 minutes.
6. You may repeat the above procedure to take the ECG Measurement.

8.4 Evaluating ECG measured values

Once you have taken the measurement, the following results may appear on the LCD display.

	ECG recording is OK.		Indicates one or more pauses in the cardiac cycle, each of which is longer than 2 seconds.
	Indicates a decreased heart rate (bradycardia) of less than 60 [bpm].		Indicates a disruption to the rhythm during the ECG recording.
	Indicates an increased heart rate (tachycardia) of greater than 100 [bpm].		

Background information and medical statistics for your doctor can be found in the “Annex for attending doctor” document included with your device.

8.5 Displaying and deleting ECG measured values

The device has two user memories. Each user memory can store up to 60 measurements. Once all memory spaces in a user memory have been used, the oldest measurements are replaced by the most recent.

You can access stored measurements on the device at any time. Proceed as follows:

1. With the device switched off, press the memory button **M**. The last stored measured value in the selected user memory is displayed.

 To access the measured values in the second user memory, press and hold the memory button **M** for 3 seconds..

2. To navigate between the individual measurements, press the memory button **M**.

If you want to delete the entire memory of the respective user, press the **M** memory button again and press and hold it together with the **START/STOP** button  for 5 seconds.

9. Transferring measurements

9.1 Transfer via *Bluetooth*[®] low energy technology

It is also possible to transfer the measured values saved on the device to your smartphone using *Bluetooth*[®] low energy technology. You will need the “beurer HealthManager Pro” / “beurer HealthManager” app for this. The app is available free of charge on the Apple App Store and from Google Play.

System requirements for the “beurer HealthManager Pro” / “beurer HealthManager” app

- iOS ≥ 12.0, Android™ ≥ 8.0
- *Bluetooth*[®] ≥ 4.0

List of compatible devices:

Proceed as follows to transfer values:

Data is transferred if *Bluetooth*[®] has been activated in the settings menu. The *Bluetooth*[®] symbol appears on the display.

	Step 1: BM 93 <i>Bluetooth</i> [®] is always activated when device is turned on.
	
	Step 2: “beurer HealthManager Pro” / “beurer HealthManager” app In the “beurer HealthManager Pro” / “beurer HealthManager” app, add the BM93 under “Settings/My devices”.
	
	Step 3: “beurer HealthManager Pro” / “beurer HealthManager” app Please follow the instructions in the app.

 The “beurer HealthManager Pro” / “beurer HealthManager” app must be active to allow data transfers.

10. Cleaning and maintenance

- Clean the main unit and cuff carefully using a slightly damp cloth only.
- Do not use any corrosive cleaning agents or solvents.
- If the surfaces of the ECG electrodes are dirty, clean them with a cotton bud soaked in rubbing alcohol.
- Under no circumstances should you hold the main unit or cuff under water, as this can cause liquid to enter and damage the device internally.
- No heavy objects should be placed on top of the device when in storage. Remove the batteries. The cuff line should not be bent sharply.

11. Accessories and replacement parts

Accessories and replacement parts are available from the corresponding service address (according to the service address list). Please state the corresponding order number.

Designation	Item number and/or order number
Universal cuff (23-43 cm)	164.226

12. What if there are problems?

Problem / Error Message	Possible cause	Solution
The blood pressure measurement result shows EE0	1. Inflating too slow or cannot inflate normally. 2. Air leakage of Cuff	Please repeat the measurement. Make sure that the cuff tube is properly inserted and that you do not move or talk. Re-insert the batteries if necessary, or else replace them.
The blood pressure measurement result shows EE1	Abnormality found in air releasing of arm cuff.	
The blood pressure measurement result shows EE2 / EE3	Abnormality found in blood pressure readings.	
The blood pressure measurement result shows EE4	Low battery.	
The ECG measurement flashes on the display.	The ECG signal is unstable or too weak.	

Problem / Error Message	Possible cause	Solution
The ECG measurement shows EE on the display.	<ul style="list-style-type: none"> - Contact pressure on the skin is too weak. - The measurement was interrupted. - The sound level was too high during the measurement. 	Repeat the measurement as per the instructions in these Instructions for use.
	The batteries are almost empty.	Replace the batteries.
The device will not switch on.	The batteries are flat.	Replace the batteries.
	The batteries have been inserted incorrectly.	Re-insert the batteries, observing the correct polarity (-/+).
The cuff does not inflate.	The cuff line is not correctly inserted into the device.	Ensure that the cuff line is correctly inserted into the device.
	The cuff is ruptured.	Replace the cuff. Contact Customer Services.
The blood pressure measurements are very high/low.	The cuff was not attached correctly.	Re-attach the cuff.
	You moved or talked during the measurement.	Do not move or talk during the measurement.
	The measurement was obstructed by clothing.	Ensure that clothing cannot obstruct the arm cuff during the measurement.

Problem / Error Message	Possible cause	Solution
Stored measured values can no longer be retrieved from the memory.	Old measured values have been overwritten by more recent values as the memory is full.	Download the stored measured values to your computer every so often.
The ECG measurement does not start even though there is contact with the skin.	Insufficient contact pressure.	Ensure that you press firmly on the electrodes.
<i>Bluetooth</i> [®] connection failed.	Connection problems between the smartphone/tablet and the app.	Switch off the main unit, close the app, and deactivate and reactivate <i>Bluetooth</i> [®] on your smartphone/tablet. Try to re-establish the connection.

13. Disposal

Repairing and disposing of the device

- Do not repair or adjust the device yourself. Proper operation can no longer be guaranteed in this case.
- Do not open the device. Failure to comply will invalidate the warranty.
- Repairs must only be carried out by Customer Services or authorised retailers. Before making a claim, please check the batteries first and replace them if necessary.
- For environmental reasons, do not dispose of the device in the household waste at the end of its useful life. Dispose of the device at a suitable local collection or recycling point in your country. Dispose of the device in accordance with EC Directive – WEEE (Waste Electrical and Electronic Equipment). If you have any questions, please contact the local authorities responsible for waste disposal.

Disposing of the batteries

- Empty, completely flat batteries must be disposed of through specially designated collection boxes, recycling points or electronics retailers. You are legally required to dispose of the batteries.

- The codes below are printed on batteries containing harmful substances:

Pb = Battery contains lead,

Cd = Battery contains cadmium,

Hg = Battery contains mercury.

14. Technical specifications

Model No.	BM 93
Measurement method	Oscillometric, non-invasive blood pressure measurement on the upper arm. Single-channel ECG in freely selectable front positions / ECG signal ground (earth) referenced.
Measurement range (blood pressure measurement)	Cuff pressure 0–300 mmHg, systolic 50–280 mmHg, diastolic 40–150 mmHg, Pulse 40–199 beats/minute
ECG bandwidth/sample rate	0.05 to 40 Hz/125 Hz
ECG Heart Rate range	5–199 beats/minute
Display accuracy	Systolic ± 3 mmHg, Diastolic ± 3 mmHg, Pulse $\pm 5\%$ of the value shown
Measurement inaccuracy	Max. permissible standard deviation according to clinical testing: Systolic 8 mmHg/diastolic 8 mmHg
Memory	2x 60 memory spaces
Dimensions	Main unit: L 128 mm x W 128 mm x H 40 mm
Weight	Main unit: Approximately 300 g (without batteries, with cuff)
Cuff size	23 to 43 cm

Permissible operating conditions	+5 °C to +40 °C, 15 to 93 % relative humidity (non-condensing)
Permissible storage and transport conditions	-25 °C to +70 °C, up to 93 % relative humidity
Power supply	4x 1.5V AAA batteries
Battery life	For approx. 240 measurements, depending on levels of blood pressure and pump pressure
Classification	Internal supply, IP21, no AP or APG, continuous operation Blood pressure: Application part, type BF ECG electrodes: Application part type CF
Data transfer via <i>Bluetooth</i> [®] wireless technology	The product uses <i>Bluetooth</i> [®] low energy technology, frequency band 2.400 - 2.483 GHz, maximum transmission power radiated in the frequency band < 20 dBm, compatible with <i>Bluetooth</i> [®] ≥4.0 smart phones /tablets

The serial number is located on the device or in the battery compartment.

Technical information is subject to change without notification to allow for updates.

- This blood pressure monitor complies with European Standard EN60601-1-2 (in accordance with CISPR 11, IEC 61000-4-2, IEC 61000-4-3 und IEC 61000-4-8) and is subject to particular precautions with regard to electromagnetic compatibility. Please note that portable and mobile HF communication systems may interfere with this unit.
- The blood pressure monitor complies with the EU Medical Devices Directive 93/42/EEC, the German Medical Devices Act (Medizinproduktgesetz) and the standards EN1060-1 (non-invasive sphygmomanometers – Part 1: General requirements), EN1060-3 (non-invasive sphygmomanometers – Part 3: Supplementary requirements for electro-mechanical blood pressure measuring systems) and IEC80601-2-30 (Medical electrical equipment – Part 2 – 30: Particular requirements for the basic safety and essential performance of automated non-invasive sphygmomanometers).
- The ECG-electrodes comply with the EU Medical Devices Directive 93/42/EEC, the German Medical Devices Act (Medizinproduktgesetz) and the standards IEC 60601-2-25 (Medical electrical equipment - part 2-25: Particular requirements for the safety of electrocardiographs) and IEC 60601-2-47

(Medical devices - part 2-47: Particular requirements for the safety, including essential performance characteristics of ambulatory electrocardiographic systems).

- The safety class of the ECG-electrodes is CF.
- The accuracy of this blood pressure monitor has been carefully checked and developed with regard to a long useful life. If using the device for commercial medical purposes, it must be regularly tested for accuracy by appropriate means. Precise instructions for checking accuracy may be requested from the service address.
- We hereby confirm that this product complies with the European RED Directive 2014/53/EU. The CE Declaration of Conformity for this product can be found under:
www.beurer.com/web/we-landingpages/de/cedeclarationofconformity.php

15. Warranty / service

Beurer GmbH, Söflinger Straße 218, 89077 Ulm, Germany (hereinafter referred to as “Beurer”) provides a warranty for this product, subject to the requirements below and to the extent described as follows.

The warranty conditions below shall not affect the seller’s statutory warranty obligations which ensue from the sales agreement with the buyer.

The warranty shall apply without prejudice to any mandatory statutory provisions on liability.

Beurer guarantees the perfect functionality and completeness of this product.

The worldwide warranty period is 5 years, commencing from the purchase of the new, unused product from the seller.

The warranty only applies to products purchased by the buyer as a consumer and used exclusively for personal purposes in the context of domestic use.

German law shall apply.

During the warranty period, should this product prove to be incomplete or defective in functionality in accordance with the following provisions, Beurer shall carry out a repair or a replacement delivery free of charge, in accordance with these warranty conditions.

If the buyer wishes to make a warranty claim, they should approach their local retailer in the first instance: see the attached “International Service” list of service addresses.

The buyer will then receive further information about the processing of the warranty claim, e.g. where they can send the product and what documentation is required.

A warranty claim shall only be considered if the buyer can provide Beurer, or an authorised Beurer partner, with

- a copy of the invoice/purchase receipt, and
- the original product.

The following are explicitly excluded from this warranty:

- deterioration due to normal use or consumption of the product;
- accessories supplied with this product which are worn out or used up through proper use (e.g. batteries, rechargeable batteries, cuffs, seals, electrodes, light sources, attachments and nebuliser accessories);
- products that are used, cleaned, stored or maintained improperly and/or contrary to the provisions of the instructions for use, as well as products that have been opened, repaired or modified by the buyer or by a service centre not authorised by Beurer;
- damage that arises during transport between manufacturer and customer, or between service centre and customer;
- products purchased as seconds or as used goods;
- consequential damage arising from a fault in this product (however, in this case, claims may exist arising from product liability or other compulsory statutory liability provisions).

Repairs or an exchange in full do not extend the warranty period under any circumstances.

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Important information regarding Electromagnetic Compatibility (EMC)

This device manufactured by MD Biomedical, Inc. conforms to IEC60601-1-2:2014 Electromagnetic Compatibility (EMC) standard. Nevertheless, special precautions need to be observed:

- The use of accessories and cables other than those specified or provided by MD Biomedical, Inc. could result in increased electromagnetic emission or decreased electromagnetic immunity of the device and result in improper operation.
- During measurement, the use of the device adjacent to or stacked with other device should be avoided because it could result in improper operation. In case such use is necessary, the device and other device should be observed to verify that they are operating normally.
- During measurement, Portable RF communications device (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the device, including cables specified by MD Biomedical, Inc. Otherwise, degradation of the performance of the device could result.
- Refer to further guidance below regarding the EMC environment in which the device should be used.

Table 1 - EMISSION Limits and Compliance

Phenomenon	EMISSION Limits	Compliance
Conducted and radiated RF EMISSIONS	CISPR 11	Group1, Class B

Table 2 - IMMUNITY TEST LEVELS

Phenomenon	Basic EMC standard	IMMUNITY TEST LEVELS
Electrostatic discharge	IEC 61000-4-2	±8 kV contact ±2 kV, ±4 kV, ±8 kV, ±15 kV air for enclosure port
Radiated RF electromagnetic fields	IEC 61000-4-3	3 V/m 80 MHz to 2.7 GHz 80 % AM at 1 kHz for enclosure port
Proximity fields from RF wireless communications equipment	IEC 61000-4-3	See table 3
Rated power frequency magnetic fields	IEC 61000-4-8	30 A/m 50 Hz and 60Hz for enclosure port

Table 3 - Test specifications for ENCLOSURE PORT IMMUNITY to RF wireless communications device

Test frequency (MHz)	Band (MHz)	Service	Modulation	Maximum power (W)	Distance (m)	IMMUNITY TEST LEVEL (V/m)
385	380 to 390	TETRA 400	Pulse modulation 18 Hz	1.8	0.3	27
450	430 to 470	GMRS 460, FRS 460	FM ±5 kHz deviation 1 kHz sine	2	0.3	28
710	704 to 787	LTE Band 13, 17	Pulse modulation 217 Hz	0.2	0.3	9
745						
780						

Test frequency (MHz)	Band (MHz)	Service	Modulation	Maximum power (W)	Distance (m)	IMMUNITY TEST LEVEL (V/m)
810	800 to 960	GSM 800/900, TETRA 800, iDEN 820, CDMA 850, LTE Band 5	Pulse modulation 18 Hz	2	0.3	28
870						
930						
1720	1700 to 1990	GSM 1800; CDMA 1900; GSM 1900; DECT; LTE Band 1, 3, 4, 25; UMTS	Pulse modulation 217 Hz	2	0.3	28
1845						
1970						
2450	2400 to 2570	Bluetooth®, WLAN, 802.11 b/g/n, RFID 2450, LTE Band 7	Pulse modulation 217 Hz	2	0.3	28
5240	5100 to 5800	WLAN 802.11 a/n	Pulse modulation 217 Hz	0.2	0.3	9
5500						
5785						

The *Bluetooth*[®] word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. and any use of such marks by Beurer GmbH is under license. Other trademarks and trade names are those of their respective owners.

Apple and the Apple logo are trademarks of Apple Inc., registered in the U.S. and other countries. App Store is a service mark of Apple Inc., registered in the U.S. and other countries.

Google Play and the Google Play logo are trademarks of Google LLC.

Android is a trademark of Google LLC.

MD Biomedical, Inc. 8F., No. 222, Sec 4, Chengde Road, Taipei City 111, Taiwan

MDSS Medical Device Service Safety GmbH, Schiffgraben 41, 30175 Hannover, Germany

Distributed by: BEURER GmbH, Söflinger Str. 218, 89077 Ulm (Germany)
www.beurer.com, www.beurer-blutdruck.de
www.beurer-gesundheitsratgeber.com, www.beurer-healthguide.com

 1639