PICCOLO AUDIOMETER

QUICK START GUIDE

EN – Quick Start Guide

IT – Guida Rapida

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NL – Snelgids

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QUICK START GUIDE: ENGLISH

INTENDED USE

The Piccolo audiometer is designed for use by audiology specialists conducting pure tone and speech audiometry examinations. Audiometric tests must be conducted in a particularly quiet environment, and preferably in a soundproof booth. If these conditions cannot be provided, the patient should be given sound-isolating headphones to wear, so that ambient and background noise can be blocked out as far as possible.

PRECAUTIONS

Users are entirely responsible for any malfunction attributable to improper use, or to maintenance or repair operations that have been carried out by persons or providers other than Inventis srl or an authorized Service Centre.

Installation

The Piccolo audiometer is a medical device. Any other external device to which it is connected within the “patient area” must also be a medical device, or must be protected by an isolating transformer.

Avoid installing and using the Piccolo audiometer close to sources of strong electromagnetic fields, as these could interfere with the operation of the device.

If the Piccolo audiometer is powered only by way of a USB cable, maximum values effectively obtainable (in AC and BC) are 10 dB lower than the maximum values nominally obtainable.

Calibration

The calibration is valid only for the transducers supplied with the device. If a transducer is replaced, the audiometer must be recalibrated.

Use

The audiometer can deliver tones at intensities that could potentially damage the patient’s ear. Be certain to set the stimulus at a suitable intensity before presenting it to the patient.

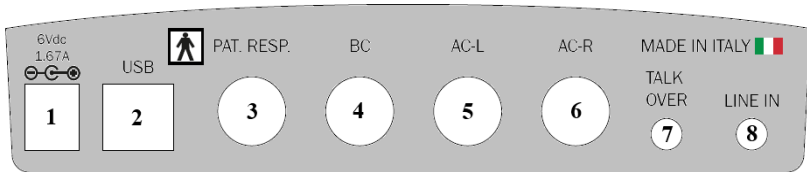
The eartips of ER-3A and ER-5A insert earphones and the masking earphones are disposable: do not use the same tips for different patients. Dispose of eartips after use.

Disinfect the cushions of headphones between one patient and the next.

REAR PANEL CONNECTIONS

Rear Panel of Piccolo Speech audiometer.

Piccolo Basic and Plus audiometers do not have some of these connectors.

Connector		Description
1	6Vdc 1.67A 	Power supply. When the Piccolo is connected to a computer USB port, the power supply will not be needed
2	USB	USB port for connection to PC
3	PAT. RESP.	Patient response switch
4	BC	Bone vibrator
5	AC-L	Left headphone
6	AC-R	Right headphone
7	TALK OVER	Microphone for operator to patient communications
8	LINE IN	External line for connection of audio source when using speech audiometry

FRONT LED

The green LED near the  symbol indicates that the audiometer is powered up either from the mains power adapter or by way of a USB cable connected to a computer.

The LED near the  symbol indicates the status of communications between the audiometer and computer or iPad. This LED lights up if the audiometer is communicating with its control device. The LED can assume two different colours: green when communicating with a PC (USB cable), and blue when communicating with an iPad (Bluetooth connection).

USING THE PICCOLO WITH A COMPUTER

Installation

- The Piccolo audiometer is connected to one of the USB ports on the computer using the cable supplied (standard USB A/B cable). The connection is plug-and-play, with no special drivers required for installation purposes: a few seconds after plugging in, the operating system will recognize the device automatically.
- The Piccolo audiometer can be controlled using Daisy, the Inventis software application for managing electronic medical records, or using Noah. In either case, the audiometer is controlled by the *Maestro* module, available in versions for both Daisy and Noah. These software applications can be installed simply by inserting the CD entitled “Software Suite” (supplied with the audiometer) into the drive of the computer. After a few seconds the *Inventis Installer* should open; if this does not happen for any reason, launch the “InventisInstaller” application manually from the CD. Click on the link corresponding to the software that is going to be used (Daisy, or the Noah Maestro module) and follow the steps of the installation procedure. If Daisy is the application required, a prompt will appear during the installation procedure asking which of the modules to install: select Maestro only.

Conducting the test

To conduct audiometric tests with the Piccolo, the Maestro module must first be opened. This is done simply by clicking the Maestro button at the bottom left of the Daisy screen (for a detailed description of the Daisy application, refer to the relative manual, which will be found on the “Software Suite” CD).


Given that the Maestro modules for Daisy and Noah are broadly similar, the following description can be referred to both modules.

Once the Maestro module has been opened, the pure tone or speech audiometry window can be opened simply by selecting “Test Type” at the left of the screen.

The bottom section of the Maestro module is occupied by the control panel, used by the operator to set the various parameters (intensity, frequency, transducer) and present the stimulus to the patient. The control panels for pure tone and speech audiometry are different. These operations can be performed using either the mouse or the computer keyboard.

Pure tone audiometry

The illustration below shows the control panel for a pure tone audiometry test. On the left hand side of the panel are the stimulus controls, whereas the controls on the right are those used to set the masking signal, which is always contralateral. The stimulus can be pure tone or warble tone, continuous or pulsed; the selection is made by clicking the relative label under “Stimulus”. Two types of test can be conducted: HL (Hearing Level) or UCL (Uncomfortable level).

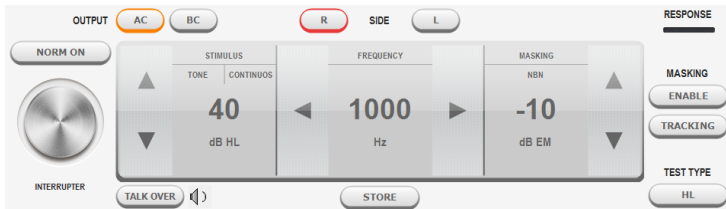Speech audiometry

Speech audiometry tests can be conducted using:

- speech material on a CD, connecting the audio source to the LINE IN input of the Piccolo;
- speech material stored on the computer (.wav files); this can be *indexed* (using Inventis ATIT software present on the CD provided); in other words a table can be created and associated with each audio track, describing the content, so that the operator can see the word and its pronunciation on the monitor of the computer. Using Daisy Maestro, the speech material must be saved in the folder c:\Daisy\Application\OMMaestro\Material. Using Noah Maestro, on the other hand, the speech material must be saved to C:\Program Files (x86)\Inventis\NoahMaestro\Material;
- the operator’s own voice (*live* speech), connecting the microphone to the TALK OVER input of the Piccolo.

The input source, and the type of masking, are selected by clicking on the relative label, above the indication of the intensity value.

The Piccolo Speech audiometer can be used to conduct different types of speech test: Word Recognition Score (WRS), Speech Reception Threshold (SRT), Most Comfortable Level (MCL), and Uncomfortable Level (UCL).

USING THE PICCOLO WITH AN IPAD

The Piccolo audiometer can be operated with an iPad using the Maestro application, which is available free of charge from the Apple App Store (enter “Inventis Maestro” as the search string).

Bluetooth connection

To allow communication between iPad and Piccolo audiometer, the Bluetooth connection must be enabled (go to the *Settings* menu and select *Bluetooth Communication*). For the iPad to recognize the Piccolo and establish communication, the user must enter the serial number (SN, an alphanumeric code comprising 13 characters), which is indicated on the label at the bottom, in the relative field of the *Bluetooth Communication* window. This operation is required only when connecting the audiometer to the iPad for the first time.

Archive window

When started, the app will show the patient archive. To add a new patient to the list, touch the  icon and enter the details of the individual (first name, surname, sex and date of birth are required).

Three more function buttons are located at the top of this screen. The  button (on the left) opens the *Settings* menu, whereas the  button is used to access the *Print/Export* menu. Finally, the “New Exam” button on the right opens a window of the same name (pure tone or speech).

Export and print exams

The export exams function generates a file of *.nha* format that can be imported into Noah. The user can export the exams of just one patient, or all of those contained currently in the archive. The file is either e-mailed to the *default* recipient or saved in the *Maestro documents* folder, depending on which option is selected.

It is also possible to generate a report in pdf format, and to print a report on an AirPrint compatible printer.

Pure tone audiometry

The bottom part of the exam window is occupied by the control panel of the audiometer (pictured below); the functions are the same as those of a traditional audiometer. More exactly, on the left hand side of the panel are the stimulus controls, whereas the controls on the right are those used to set the masking signal, which is always contralateral. The stimulus can be pure tone or warble tone, continuous or pulsed; the selection is made by clicking the relative label under “Stimulus”.

The Piccolo audiometer can be used to conduct HL or UCL type tests.

Speech audiometry

The speech audiometry test can be conducted using either the operator's own voice (*live speech*), or pre-recorded speech material stored on the iPad or provided by another audio source (such as a CD drive, for example). The input source and the type of masking are selected by touching the relative label, above the indication of the intensity value.

In the case of *live speech*, a microphone must be connected to the TALK OVER input of the Piccolo audiometer. If recorded material is to be used, on the other hand, the audio source (iPad or CD drive) must be connected to the LINE IN input of the Piccolo.

Speech material must be stored in *.wav* or *.mp3* format, and can be uploaded to the iPad from the computer. Detailed instructions for transferring speech material to the iPad are given in the full Piccolo manual.

The Piccolo Speech audiometer can be used to conduct different types of speech test: Word Recognition Score (WRS), Speech Reception Threshold (SRT), Most Comfortable Level (MCL), and Uncomfortable Level (UCL).

Saving an exam

Having completed the pure tone or speech test, pressing the Archive label at the top left will open the "Save Exam" menu, which allows the operator to select a patient name with which to associate the test just conducted, or to delete the test.

TECHNICAL SPECIFICATIONS

Classification

Piccolo Basic. Pure-tone audiometer: Type 4 (EN 60645-1 / ANSI S3.6)

Piccolo Plus and Speech. Pure-tone audiometer: Type 3 (EN 60645-1 / ANSI S3.6)

Available signals

Stimulus: pure tone, warble tone

Masking: NBN; WN and SN (only Piccolo Speech)

Speech audiometry inputs (only Piccolo Speech): EXT1, EXT2, MIC.

Signal specifications

Attenuator step: 5dB

Presentation: Continuous, Pulsed (0.5, 1, 2 Hz)

Warble: 5 Hz sin wave modulating signal

Available outputs and transducers

AC: TDH-39 headphones, ER-3A or ER-5A insert earphones

BC: B-71 bone vibrator

Pure tone: available frequencies and maximum levels (dB HL) (*)

Freq. (Hz)	AC TDH39	AC ER-3A	AC ER-5A	BC
125	80	90	90	-
250	100	105	100	45
500	110	110	110	65
750	115	115	120	70
1,000	120	120	120	75
1,500	120	120	120	80
2,000	120	120	115	80
3,000	120	120	115	75
4,000	120	110	110	75
6,000	110	95	100	55
8,000	100	90	90	50

(*) The levels indicated are for a device connected to the a.c. mains power supply. Using a USB power source, maximum levels will be 10 dB lower.

Speech audiometry: maximum levels (dB HL) (*)

AC: TDH39	AC: ER-3A	AC: ER-5A	BC
100	100	100	55

(*) The levels indicated are for a device connected to the a.c. mains power supply. Using a USB power source, maximum levels will be 10 dB lower.

Available tests

	<i>Basic</i>	<i>Plus</i>	<i>Speech</i>
Pure tone audiometry	•	•	•
Speech audiometry	-	-	•

Calibration

Validity: 12 months

TDH 39: EN ISO 389-1; *ER-3A* and *ER-5A:* EN ISO 389-2; *B71:* EN ISO 389-3

Power supply

Consumption: 8 Watts; *Power supply:* 6V, 1.67A cont., through an external medical grade 100-240 Vac 50/60 Hz power supply.

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