DEUTSCH

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ENGLISH

beurer medical FT 15/1 CE 0344

Important safety information – retain for later use.
In order to make optimum use of all the merits of the clinical thermometer, you should carefully read the operating instructions before using it, retain the instructions for later use and also keep them accessible for other users.

- The thermometer is intended to measure the temperature of the human body.

- Children should not use the thermometer without supervision.

- Before each use, check the thermometer for signs of damage or wear. Damaged or worn thermometers should not be used.

- Always wait for the acoustic signal which indicates the minimum time required for measurement. Discuss the temperature with your family physician.

- The thermometer contains sensitive electronic parts. Therefore, protect it from blows, bending, high temperature or direct exposure to sunlight.

- This thermometer has a flexible measuring tip to provide greater comfort and safety during measurement, particularly in infants and persons who are asleep or have a reduced level of consciousness. The tip must not be bent more than 90°!

- Using the thermometer in strong electromagnetic fields such as next to a mobile phone, can cause a malfunction.

- When the thermometer is switched on, it carries out a self-test.

- Therefore the accuracy of measurement need not be tested at regular intervals.

- If the temperature is very abnormal, please contact your family doctor promptly.

- This unit is in line with European Standard EN60601-1-2 and is subject to particular precautions with regard to electromagnetic compatibility (EMC). Please note that portable and mobile HF communication systems may interfere with this unit. For more details, please contact customer service at the address indicated.

Measuring methods:**Temperature measurement in the anus (rectal)**

This method of measurement is most reliable and most accurate. This method of measurement is most reliable and most accurate.

Guide the tip of the thermometer carefully 2 to 3 cm into the anus. The measurement time with this thermometer at this site is only about 10 seconds. The end of the measuring time is indicated by an signal tone.

Measurement in the mouth cavity (oral)

For this purpose, guide the tip of the thermometer carefully into one of the heat pouches beneath the tongue, to the left or the right of the root of the tongue.

Measurement of temperature in the armpit (axillary)

This method of measurement is relatively inaccurate. Therefore, it cannot be recommended from a medical point of view.

Mode of measurement Recommended time required for measurement Mean divergence to the rectal method

Mode of measurement	Recommended time required for measurement	Mean divergence to the rectal method
in the anus	until the acoustic signal	-
in the oral cavity	until the acoustic signal	-0,4 °C to -1,5 °C (-0,7 °F to -2,7 °F)
in the armpit	minimum 10 minutes	-0,7 °C to -2,0 °C (-1,26 °F to -3,6 °F)

Use

Press the On/Off switch to turn the unit on. A short acoustic signal will announce that it is switched on. The thermometer will carry out a self-test for about 2 seconds. During the self-test all segments of the display will be visible. Then »Lo °C (Lo °F)« or the value of the last measurement will appear in the display. Then the reference value of 37°C (98,6°F) will appear. Then the flashing symbol »C« appears and the symbol »Lo« is shown in the display screen. The thermometer is now ready for use. During the measurement, the current temperature is displayed continuously, and the »C« (»F«) symbol flashes on and off. The measurement is completed when a stable temperature has been reached. An acoustic signal (10 long beeps) sounds, the »C« (»F«) symbol stops flashing and the measured temperature is displayed. So long as the temperature is less than 32 °C (89,6°F), the »C« (»F«) symbol continues to flash and »Lo °C (Lo °F)« is shown on the display. Higher values are displayed

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of the following disinfectants in accordance with the manufacturer's instructions. Other cleaning agents or methods could impair the function of the thermometer or damage it.

Name of the agent: Isopropanol 70% Pharmacy; Gigasept FF; Lysoformin; Aseptisol.

Storage/Disposal

When you are not using the thermometer, please retain it in the original protective cover. The device must be tested for accuracy (calibrated) after 2 years. Please send the complete device to the dealer's or service address.

Please dispose of the device in accordance with the directive 2002/96/EG – WEEE (Waste Electrical and Electronic Equipment). If you have any queries, please refer to the local authorities responsible for waste disposal.

Technical data

Type: Maximum thermometer type FT 15/1

32 °C (89,6°F) to 42,9 °C (109,2°F)

±0,1 °C in a water bath with a

temperature of 35,5 °C (95,9°F) to

42,0 °C (107,6°F)

±0,2 °C in a water bath with a

temperature of 32,0 °C (89,6°F) to 35,4 °C (95,7°F)

±0,2 °C in a water bath with a

temperature of 42,1 °C (107,8°F) to 42,9 °C (109,2°F)

to 40 °C (122°F), with 30% to 85%

relative humidity

-10 °C (14°F) to +60 °C (140°F), with 25% to 90%

relative humidity

EN 12470-3: 2000 Clinical Thermometers

Reference of standardisation: Attention - read the operating instructions.

Explanation of symbols: Application part of Type BF

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